amieamed

EXCEED
YOUR EXPECTATIONS

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Skin Quality – A Holistic 360° View: Consensus Results

Kate Goldie^{1,*}
 Martina Kerscher^{2,*}
 Sabrina Guillen Fabi³
 Cyro Hirano⁴
 Marina Landau⁵
 Ting Song Lim⁶
 Heather Woolery-Lloyd⁷
 Kavita Mariwalla⁸
 Je-Young Park⁹
 Yana Yutskovskaya¹⁰

¹European Medical Aesthetics Ltd, London, UK; ²Division of Cosmetic Science, Department of Biochemistry and Molecular Biology, University of Hamburg, Hamburg, Germany; ³Department of Dermatology, University of California, San Diego, CA, USA; ⁴Division of Dermatology General Polyclinic of Rio de Janeiro, Private Practice, CD Clinica Dermatologica, Rio de Janeiro, Brazil; ⁵Dermatology Unit, Wolfson Medical Center, Holon, Israel; ⁶Clique Clinic, Kuala Lumpur, Malaysia; ⁷Department of Ethnic Skin Care, University of Miami Cosmetic Medicine and Research Institute, Miami Beach, FL, USA; ⁸Mariwalla Dermatology, New York, NY, USA; ⁹Apliko-Jung Department, Oracle Dermatology Center, Seoul, Republic of Korea; ¹⁰Dermatovenerology and Cosmetology Department, Pacific State Medical University of Health, Moscow, Russia

*These authors contributed equally to this work

Correspondence: Martina Kerscher
 Division of Cosmetic Science,
 Department of Biochemistry and
 Molecular Biology, University of Hamburg,
 Hamburg, Germany
 Email: martina.kerscher@uni-hamburg.de

Introduction: Skin quality is an important component of human attractiveness. To date, there are no standardized criteria for good skin quality. To establish a consensus for good skin quality parameters and measurement and treatment options, a virtual skin quality advisory board consisting of a global panel of highly experienced aesthetic dermatologists/aesthetic physicians was convened.

Methods: A total of 10 dermatologists/aesthetic physicians served on the advisory board. A modified version of the Delphi method was used to arrive at consensus. Members accessed an online platform to review statements on skin quality criteria from their peers, including treatment and measurement options, and voted to indicate whether they agreed or disagreed. Statements that did not have agreement were modified and the members voted again. Consensus was defined as: strong consensus = greater than 95% agreement; consensus = 75% to 95% agreement; majority consent = 50% to 75% agreement; no consensus = less than 50% agreement.

Results: There was strong consensus that good skin quality is defined as healthy, youthful in appearance (appearing younger than a person's chronological age), undamaged skin and that skin quality can be described across all ethnicities by four emergent perceptual categories (EPCs): skin tone evenness, skin surface evenness, skin firmness, and skin glow. The EPCs can be affected by multiple tissue layers (ie, skin surface quality can stem from and be impacted by deep structures or tissues). This means that topical approaches may not be sufficient. Instead, improving skin quality EPCs can require a multilayer treatment strategy.

Conclusion: This global advisory board established strong consensus that skin quality can be described by four EPCs, which can help clinicians determine the appropriate treatment option(s) and the tissue or skin layer(s) to address. Skin quality is important to human health and wellbeing and patients' perception for the need for aesthetic treatment.

Keywords: aesthetic treatment, consensus, emergent perceptual skin quality categories, EPCs, skin quality

Introduction

Skin quality is important to human attractiveness^{1,2} and it helps determine the need for and choice of aesthetic treatment.³ Skin quality significantly influences perception of age, attractiveness, health and youth.^{2,4,5} Even small changes in skin surface and pigmentation pattern can have a strong impact on perceived facial attractiveness.² This holds true across ethnicities and for both female and male faces.^{6–8}

Age-related skin changes stem from aging-induced alterations that occur in all tissue layers. Skeletal aging leads primarily to a loss of bony support. This is visible

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ORIGINAL RESEARCH

Safety and Effectiveness of an Automated Microneedling Device in Improving Acne Scarring

ABSTRACT

OBJECTIVE: To assess the effectiveness of the amiea med (amiea med, MT.DERM GmbH, Berlin, Germany) automated microneedling device in reducing facial atrophic acne scars.

STUDY DESIGN: Open label, single center.

by DIRK FREIHERR VON DALWIG-NOLDA, MD and GLYNIS ABLON MD, FAAD
Dr. Freiherr von Dalwig-Nolda is with the DermaVen Center in Bad Salzfluren, Germany. Dr. Ablon is with the University of California, Los Angeles in Los Angeles, California and the Ablon Skin Institute & Research Center in Manhattan Beach, California.

J Clin Aesthet Dermatol. 2020;13(8):17-22

ORIGINAL RESEARCH

Safety and Effectiveness of an Automated Microneedling Device in Improving the Signs of Aging Skin

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
ABSTRACT

Objective: The purpose of this study was to evaluate the efficacy and safety of an automated microneedling device (Exceed Amiea Med


by GLYNIS ABLON, MD, FAAD
Dr. Ablon is an Assistant Clinical Professor at the University of California in Los Angeles, California.

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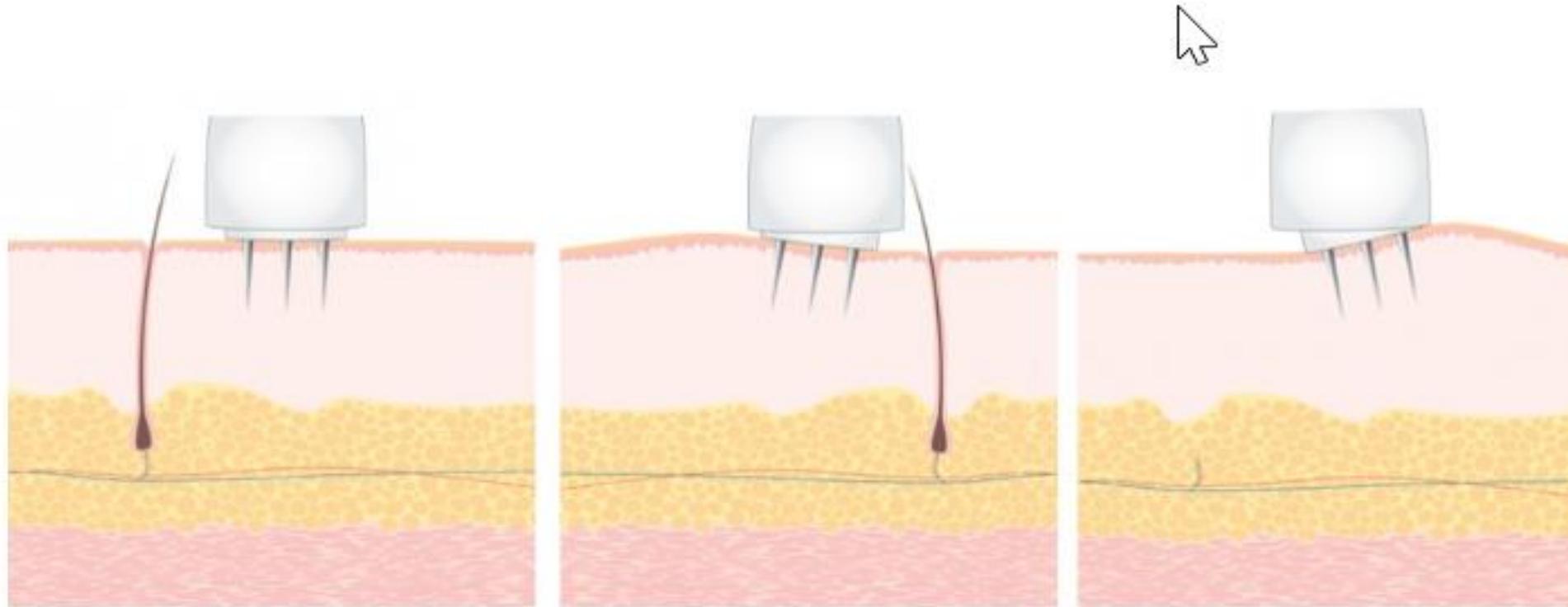
Patented tilting needle plate adapts to surface of treated skin area similar to an electric razor. Perpendicular opening of the skin barrier. No tearing of or injury to the skin's surface.

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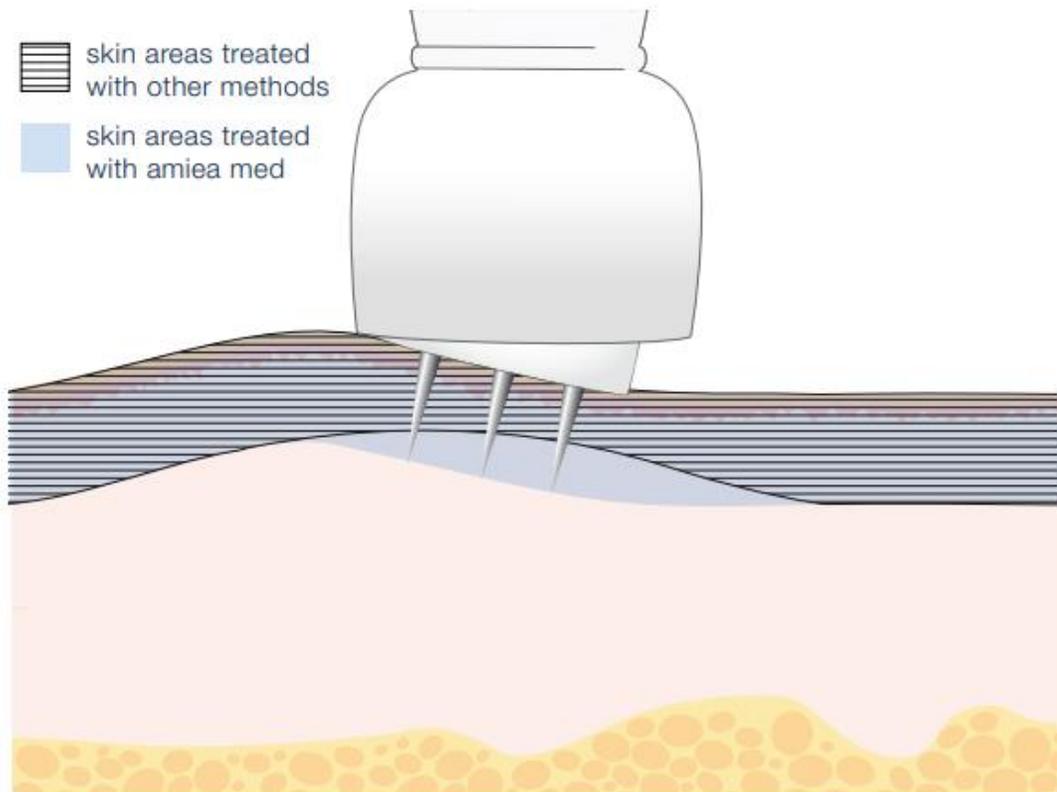All areas are easily accessible due to the tilting needle plate

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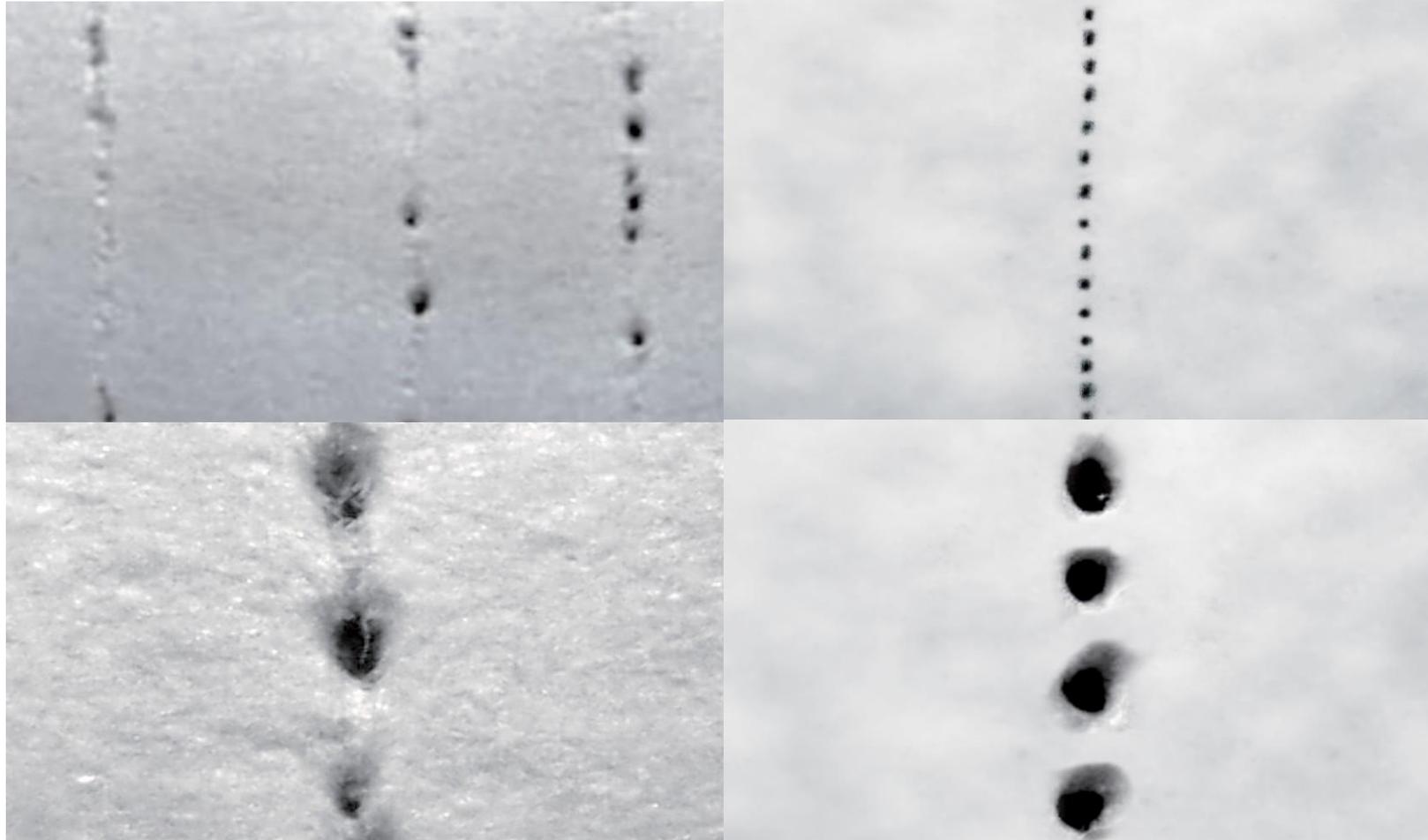
Another system
Uneven punctures, different distance

EXCEED
Even, small punctures, same distance,
perpendicular

PROFONDEUR RÉGLABLE - VERSATILE

Plastic and Reconstructive Surgery April 2008

Percutaneous Collagen Induction
Therapy:

An Alternative Treatment for Scars,
Wrinkles, and Skin Laxity. (**Matthias
Aust, M.D. Des Fernandes, M.D.**)

Similar results regardless of 1mm or
3mm.

**Micro Needling and Injecting Platelet
Rich Plasma to Enhance Collagen
Synthesis and Skin Tightening** (January
16, 2008 ~ July 16, 2008) Joseph F.
Greco M.D., New collagen forms no
deeper than 0.6 mm regardless of depth
of injury.

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